


# Auscultation Training Model

# BT-CECS

## Auscultation Training Model

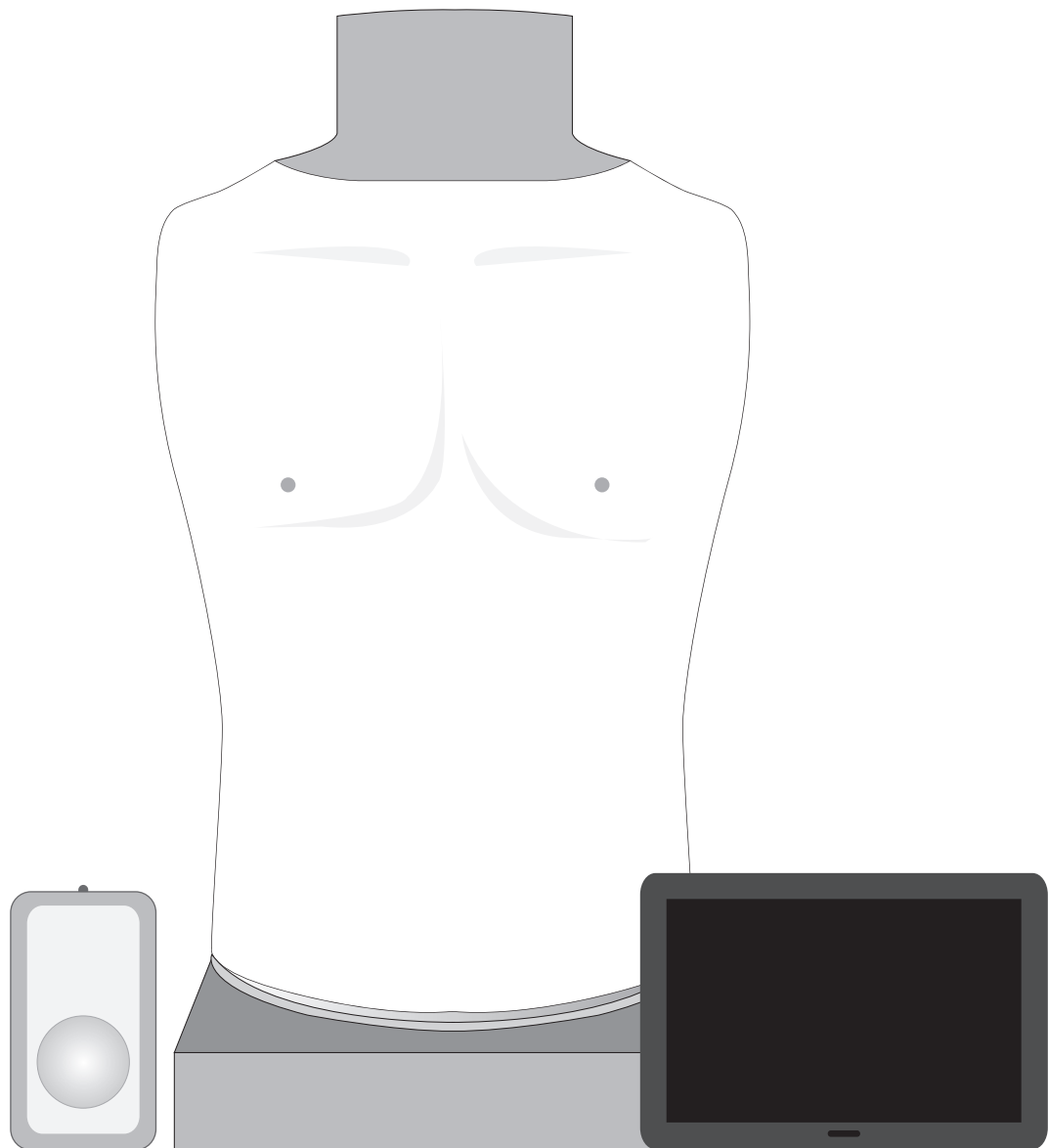

## Product Information

Product Name Auscultation Training Model  
Model Name BT-CECS  
Manufacturer BT Inc.  
Country of manufacture Republic of Korea

## Product Specifications

Dimensions 65 X 24 X 35 (cm)  
Weight 6.7kg

## Customer Service

Website [www.btinc.co.kr](http://www.btinc.co.kr)  
Address A-313, Samsung Techno-valley, 140, Tongil-ro,  
Deogyang-GU, Goyang-Si, Gyeonggi-do, 10594,  
Republic of Korea  
Telephone +82-2-2038-2903  
E-mail [bt@btinc.co.kr](mailto:bt@btinc.co.kr)



## Cautions for battery disposal


※ Please follow the regulations of corresponding area for the disposal of used batteries.

# Cautions


Please read carefully to ensure correct usage and user's safety through understanding of the product.

- ① Use only components provided or approved by BT Inc.
- ② Do not use the product for purposes other than described in the manual.
- ③ If you disassemble or modify the product without any permission by BT Inc, the warranty will be void. Also, making imitated or altered products for commercial purposes are prohibited by law.
- ④ If the product is overheating or emits a burning odor during usage, turn off the power switch, unplug the power adaptor and call the customer service center.
- ⑤ Keep the product away from water and moisture area.
- ⑥ Do not use the product near strong magnetic fields or high voltage power lines.
- ⑦ Do not use the product and unplug the power adapter during lightning storms.  
It can cause malfunction of the product and increase the risk of an electric shock.
- ⑧ The product may malfunction under the extreme weather conditions.  
(Operation temperature : 0°C~40°C (32°F~104°F), Storage temperature : -10°C~50°C (14°F~122°F))
- ⑨ Do not keep the product near hot or cold environments as possible. Deterioration of product or internal damage may occur from extreme weather conditions.
- ⑩ Never clean the product with chemicals such as solvent, benzene or thinner as these may cause fire ignition or surface deterioration.
- ⑪ Do not install other programs or software to the provided tablet.  
It may cause software malfunction.
- ⑫ BT Inc. can change or adjust the product specification without any notice in order to improve the product.

# Components

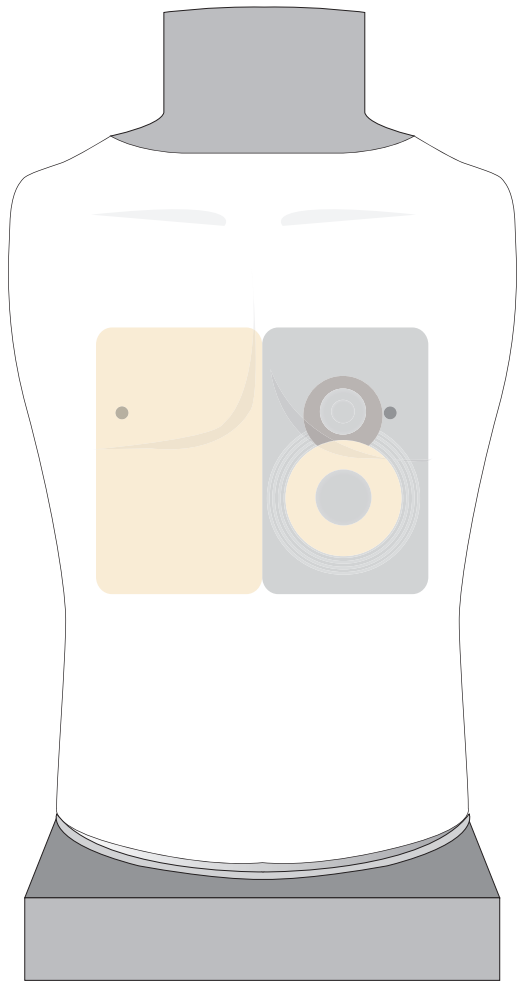

Torso model (Speaker included)

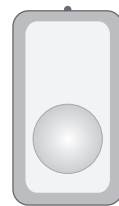

KIKUZO



AUX distribution cable



Bluetooth receiver


- Etc. – User manual, storage case
- ※ Tablet PC, Power bank – Optional

# How to register KIKUZO website


## How to create ID/PW and register license to KIKUZO website

※ Once the device is registered, customers are only able to log-in to the website with the registered device. So please make sure to register the device such as computer/laptop/smartphone which will be used with KIKUZO."


1. Connect to the website, <https://kikuzosound.com/>.
2. Click **Login** in the upper-right corner.

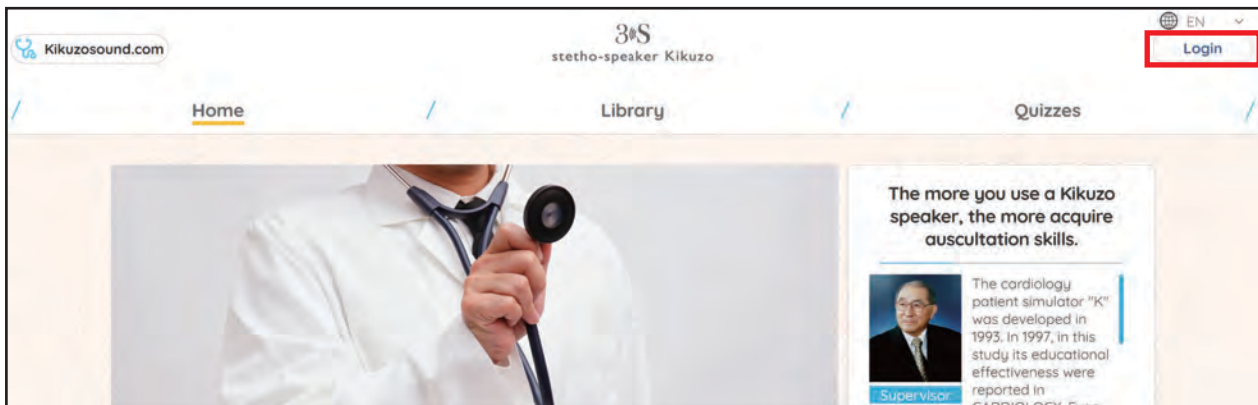

3. Scroll down the page and click "Go to Registration Page" icon.

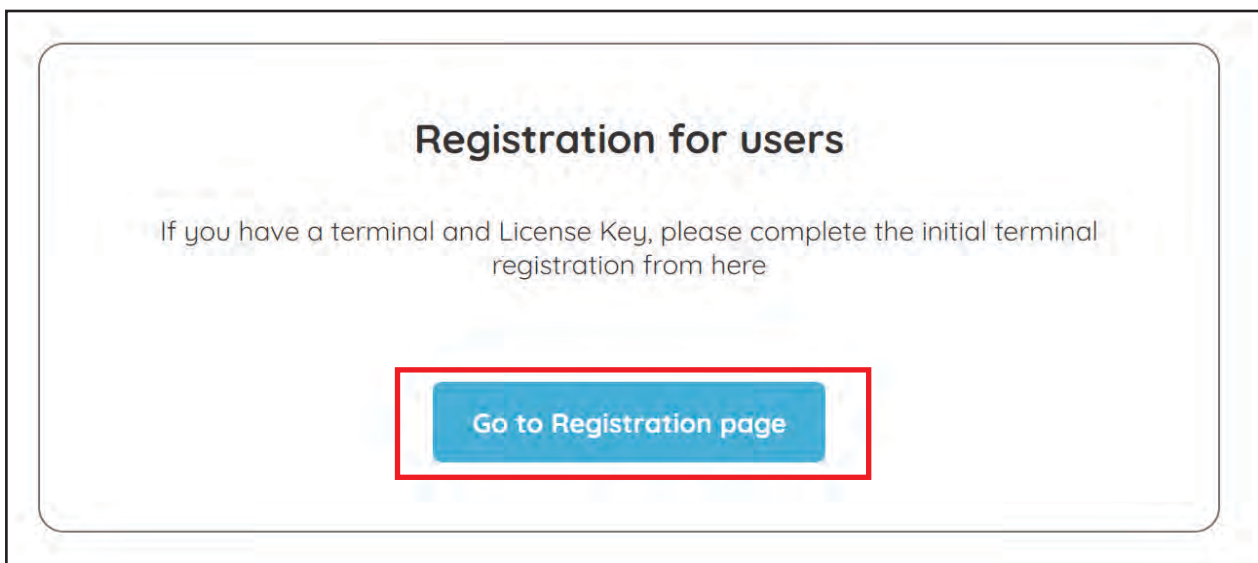

4. Click “OK” when a pop-up shows on the screen.

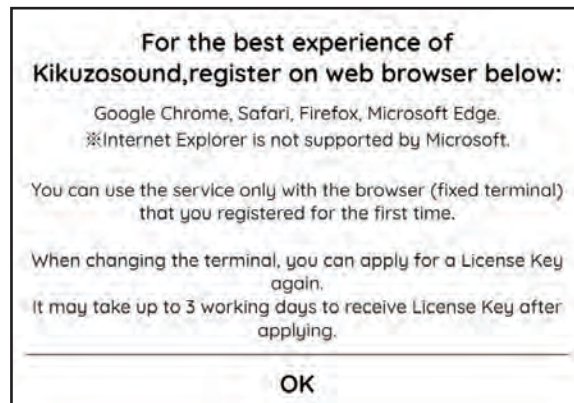

※ Available Browser : Google Chrome, Safari, Firefox, Microsoft Edge

☐ Internet Explorer is not available

※ The access to the webpage will be limited to the device that you registered the License Key, so the it will be not available to log-in with other devices.

※ If you want to change the device, please contact us, it might take up to 3 days.

5. Input **Customer Information, Kikuzosound Pass Information, Distributor Information.**

1). **Customer information** : Input all the information by referring to the picture below.

※ E-mail will be used when user authorization/re-authorization, please do not type the personal e-mail address, please describe the institution’s e-mail address which is public.

2). **Kikuzosound Pass Information** : Input the 16-digit License Key.

### Kikuzosound Pass Information

\* You can use the service only with the browser (fixed terminal) that you registered for the first time  
\* When changing the terminal, you can apply for a license key again.  
\* It may take up to 3 working days to receive license key after applying.

License Key \*Required \* Please enter the number printed on the token

--	--	--	--

3). **Distributor Information** : Input the sales representative's information for the support.  
(It is a management purpose and it is not a mandatory.)

### Distributor Information

Sales Company Name	
Contact Name	
Mail Address	
Phone Number	

4). After this, accept terms and conditions and Click “Register”.

6. You will receive an approval e-mail titled “Registration complete” with the e-mail that you typed during registration. Click the link below to complete the registration.

☆ **Registration complete** ☐

☏ 보낸사람 Kikuzosound.com <kikuzosound@telemedica.co.jp> 20.06.16 11:37 주소추가 | 수신차단

---

Thank you for using "Kikuzosound.com".


You have successfully registered a new account.

Please click the URL below to get started.


[https://kikuzosound.com/member\\_login/activate/1085-1870-1727-1360](https://kikuzosound.com/member_login/activate/1085-1870-1727-1360)

7. Feel free to use the website with the login information that you have.


# How to install the manikin

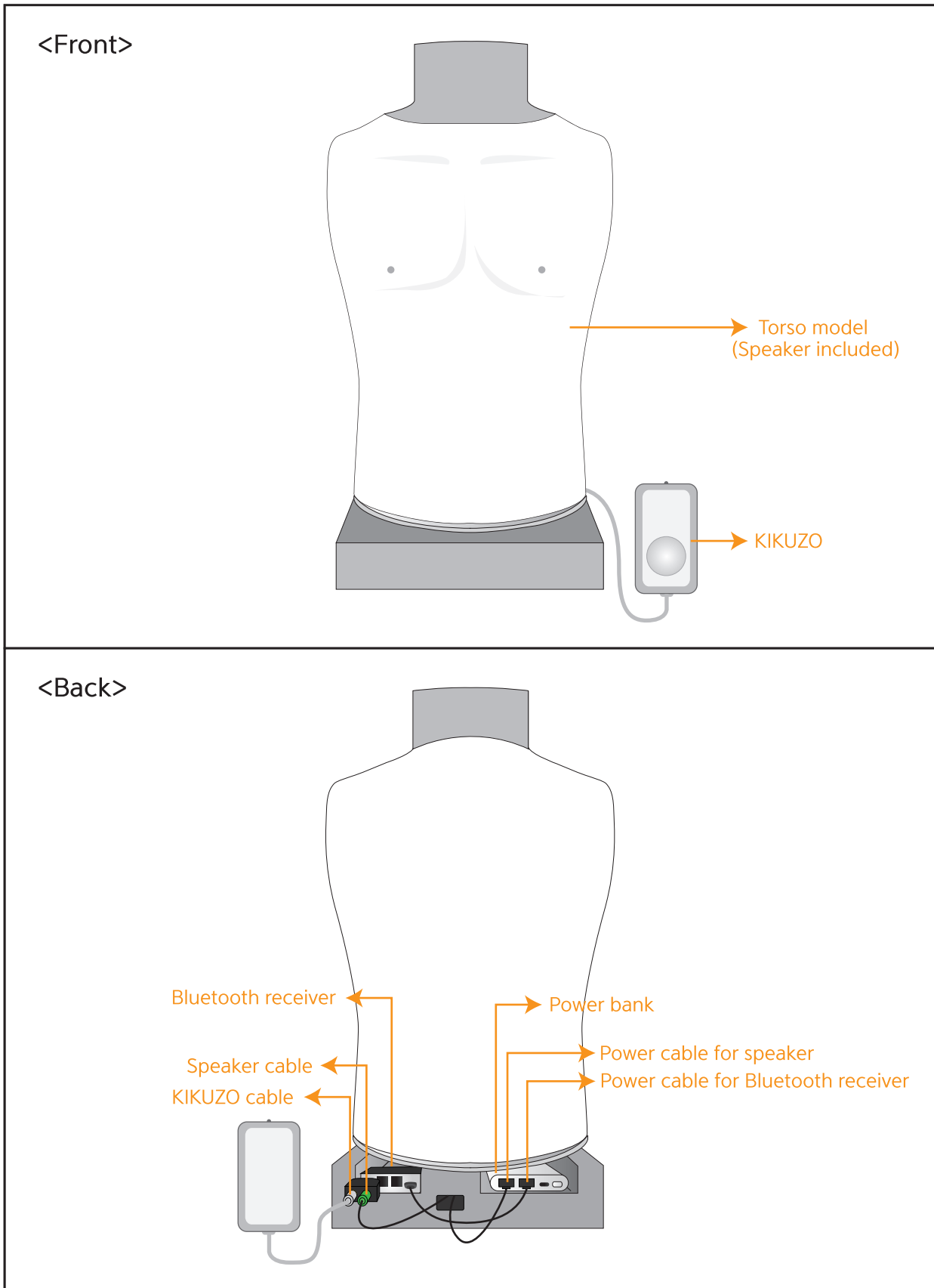





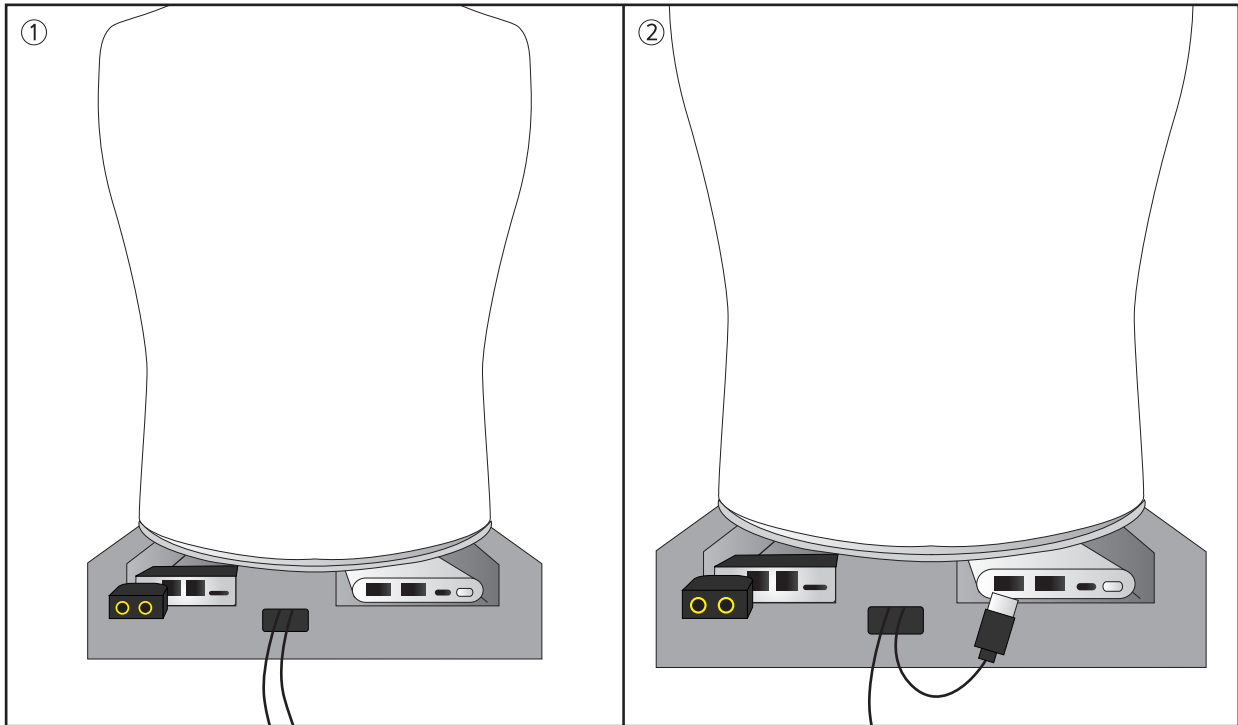

- ① Place the power cable and Bluetooth receiver at the bottom of the back of manikin.
- ② Connect the power cable of speaker to the power bank.


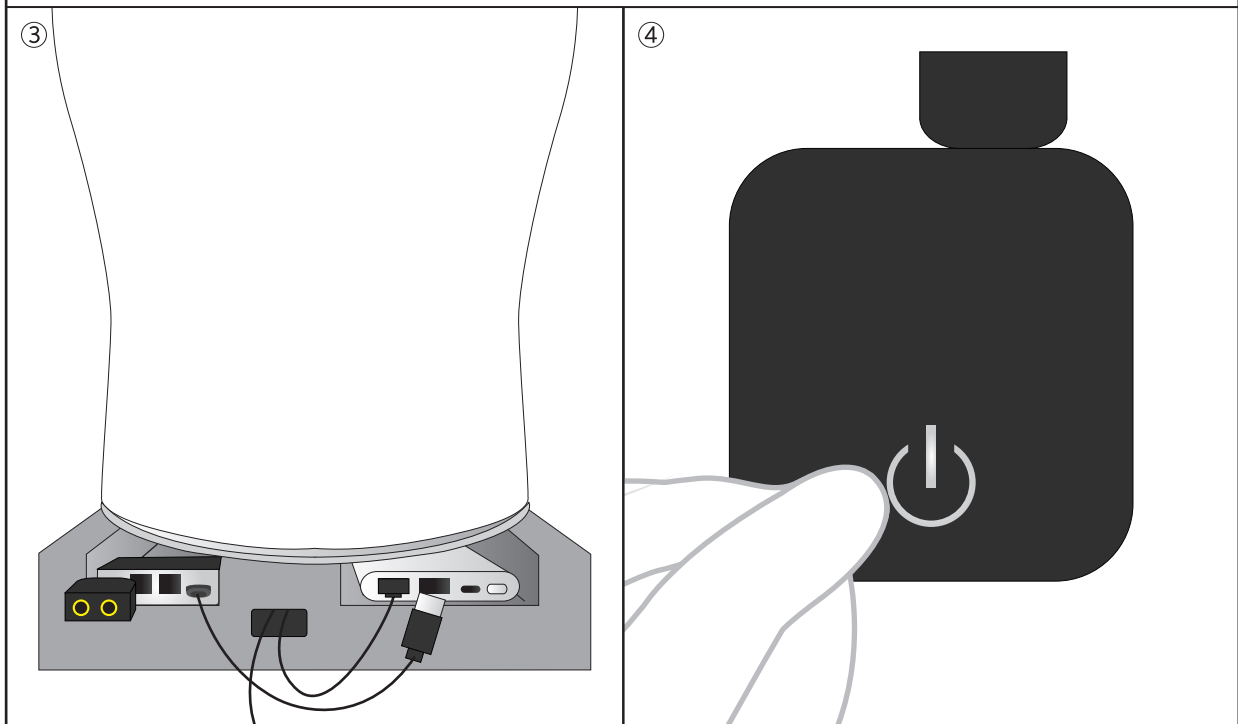


- ③ Connect the power cable of Bluetooth receiver to the power bank.
- ④ Turn on the Bluetooth receiver by pressing the power button for 3 seconds.

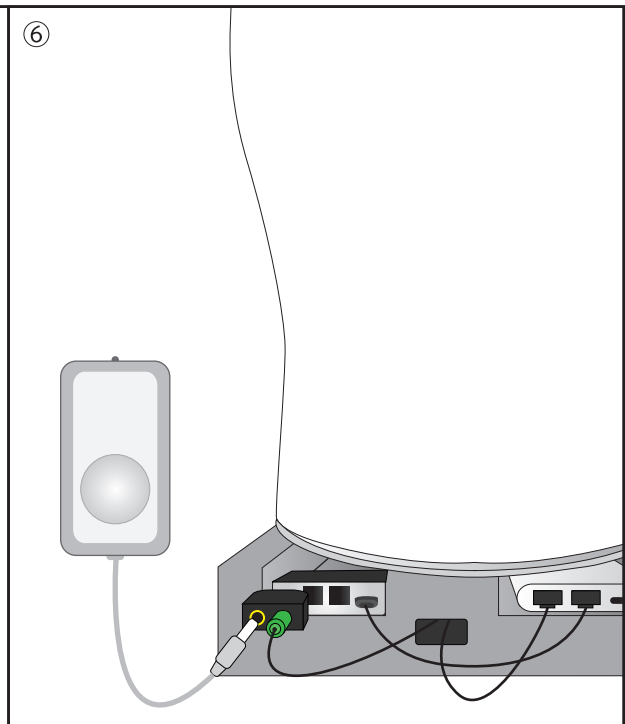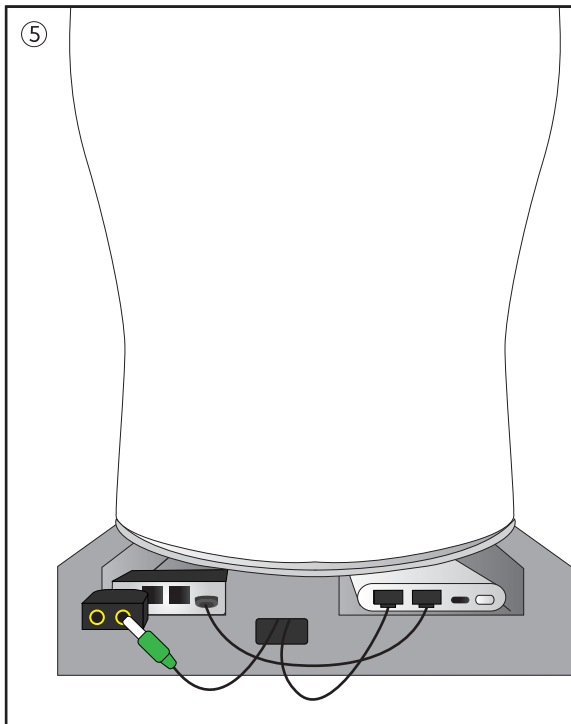

⑤ Connect the speaker cable to the Bluetooth receiver.

⑥ Connect the KIKUZO cable to the Bluetooth receiver.

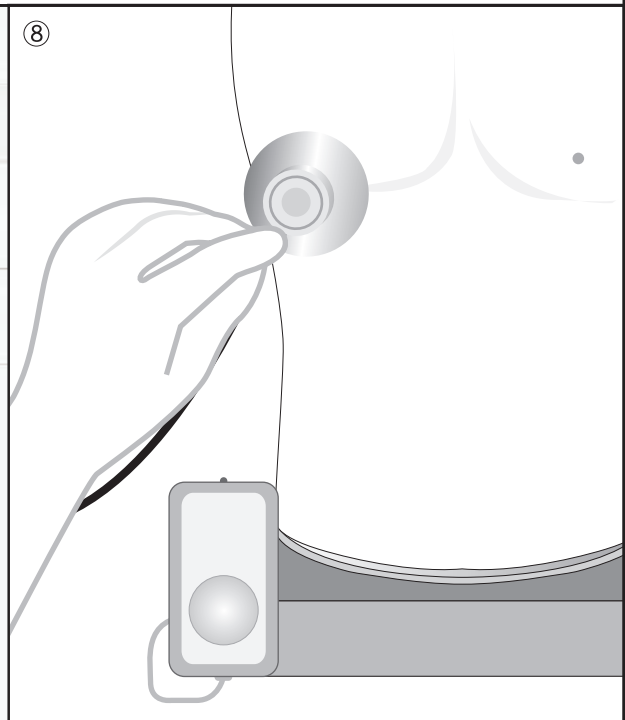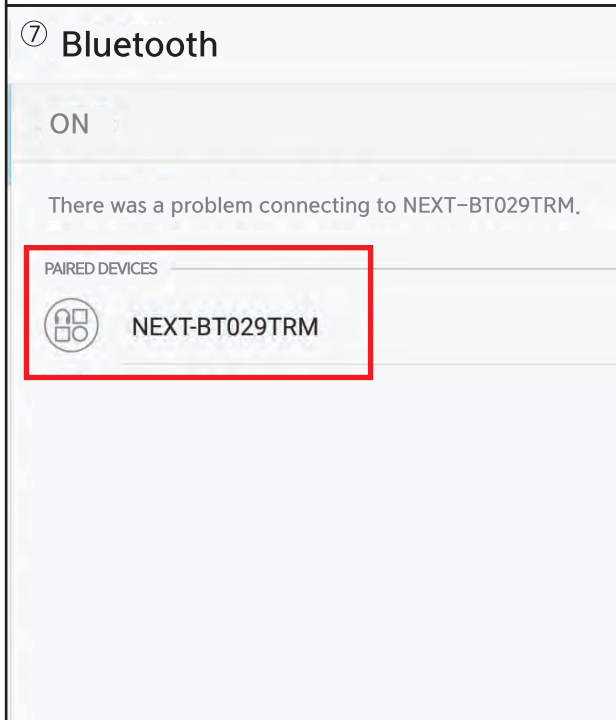

⑦ Pairing the Bluetooth receiver with the device such as tablet or laptop.

⑧ Log-in to KIKUZO website and start the training.

※ After the training, turn off the Bluetooth receiver by pressing the power button for 3 seconds and disconnect the cables from the power bank.

# Auscultation Training Model

# BT-CECS

A-313, Samsung Techno-valley, 140, Tongil-ro,  
Deogyang-gu, Goyang-si, Gyeonggi-do, 10594, Republic of Korea  
Homepage: [www.btinc.co.kr](http://www.btinc.co.kr) / E-mail: [btinc@btinc.co.kr](mailto:btinc@btinc.co.kr)  
Tel: +82-2-2038-2903 ,+82-51-903-1144 / Fax: +82 303-3130-0607

Copyright(c) BT Inc. All rights reserved.